



Instructions for Use

Finger-switch Pencil / Foot Switch-Single Use

Description/Intended Use:

These devices are single-use supplied sterile. Check plug matching on generator base and connect. Press CUT/yellow button for cutting and COAG/blue button for coagulation.

Contraindications:

These devices should never be used when:

- There is visible evidence of damage to the exterior of the device such as cracked plastic or connector damage.
- These devices fail the inspection described herein.
- In the presence of flammable gases, liquids, and/or oxygen enriched environments.

Safety Tips:

- Use lowest possible power setting on electrosurgical unit capable of achieving desired surgical effect.
- Never allow the cables connected to these devices to be in contact with skin of the patient or operator during electrosurgical activations.
- Do not permit the cables connected to these devices to be parallel and in close proximity to the leads of other electrical devices.
- Always place unused electrosurgical accessories in a safe insulated location such as a holster when not in use.
- Inspect and test each device before each use.
- Discard devices that have reached their life expectancy.
- Activation of an electrosurgical device when not in contact with target tissue or in position to deliver energy to target tissue (fulguration) may cause capacitive coupling.

Inspection:

These devices should be inspected before each use. Visually examine the devices for obvious physical damage including:

- Cracked, broken or otherwise distorted plastic parts.
- Broken or significantly bent connector contacts.
- Damage including cuts, punctures, nicks, abrasion, unusual lumps, significant discoloration.
- Tips for damage, corrosion or misalignment condition.
- On hand-switching forceps, switch contacts for corrosion, discoloration and flatness.

Warranty:

These products are guaranteed against faulty material or workmanship. The warranty is null and void should damage occur as a result of improper handling or use.

 **Attention**, see instructions for use  Latex Free  Single use  Expiry Date



Tecno, Sialkot, Pakistan

Distributed by;

Timesco of London Ltd. Timesco House, 3 Carnival Park, Carnival Close, Basildon,
Essex SS14 3WN, England.

T: +44 (0) 1268 297 700 F: +44 (0) 1268 297 800 E: info@timesco.com W: www.timesco.com

I.SI.003/1 03/10

